



Manufactured By: Church & Dwight Co., Inc., Ewing, NJ 08628
Made in Canada

Replens is a trademark of Church & Dwight Co., Inc.
REIN-00104-07
72032112

INDICATIONS FOR USE

Replens™ Long-Lasting Vaginal Moisturizer is a personal lubricant for vaginal application, intended to moisturize and lubricate, to enhance the ease and comfort of intimate sexual activity and to supplement the body's natural lubrication.

This product is compatible with natural rubber latex and polyisoprene condoms. This product is not compatible with polyurethane condoms.

DIRECTIONS FOR USE

1. Wash hands before use.
2. Remove the pre-filled applicator from the sealed wrapper.
Do not remove the tab at this time.
3. Grip the applicator firmly by the thick end. Flick the thin end of the applicator down several times to ensure contents are in the tip.
4. Twist the tab at the tip of the applicator to remove it.*
***DO NOT use scissors or sharp tools to cut the applicator open.**
5. Sitting or lying on your back with knees bent, gently insert open end of applicator well into the vagina.
6. Squeeze the thick end of the applicator firmly 2-3 times to deposit gel.
7. Remove the applicator and discard immediately in waste container.

Note: The applicator is designed to deposit the recommended amount of gel. When used as directed, a small amount of gel will remain in the applicator after use. Discard this excess product along with the applicator and do not attempt to reuse the applicator.



PLEASE READ THE FOLLOWING CAREFULLY BEFORE USE.

Warnings

• Replens™ Long-Lasting Vaginal Moisturizer is not a contraceptive and does not contain a spermicide. Keep out of eyes and ears. If vaginal irritation occurs, discontinue use or contact your physician. If pregnant or breastfeeding, contact your health care provider before use.

• KEEP OUT OF REACH OF CHILDREN.

• Store at room temperature 59-86°F (15-30°C).

TAMPER EVIDENT FEATURE:

For your protection, each applicator is individually wrapped. DO NOT USE this product if an applicator is unwrapped or the wrapper is torn or missing.

HOW DOES REPLENS™ LONG-LASTING VAGINAL MOISTURIZER WORK?


Replens™ Long-Lasting Vaginal Moisturizer is estrogen-free and provides long-lasting moisture. When you apply Replens™ Long-Lasting Moisturizer, it immediately goes to work to provide long lasting moisture for the vaginal cells to absorb. As the cells of the vaginal wall are regenerated, dead vaginal lining cells are cleared and the product is eliminated naturally. As with dry skin that you experience on your face and hands, regular moisturizing treatment may be necessary to prevent dryness from recurring.

When you apply Replens™ Long-Lasting Vaginal Moisturizer, it immediately goes to work coating dry vaginal cells. Replens™ Long-Lasting Vaginal Moisturizer continues to provide moisture for up to 3 days, allowing the dry cells to absorb the moisture and return to natural function. As the cells are regenerated, dead cells are cleared and the product is eliminated naturally, leaving healthier, more supple vaginal tissues.

Panel 1:
Replens™ Long-Lasting Vaginal Moisturizer coats to provide immediate relief to dry vaginal lining cells

Panel 2:
Replens™ Long-Lasting Vaginal Moisturizer delivers continuous moisture which the cells absorb, regaining natural integrity and elasticity

Panel 3:
Dead vaginal lining cells are eliminated and skin is left moisturized and rejuvenated

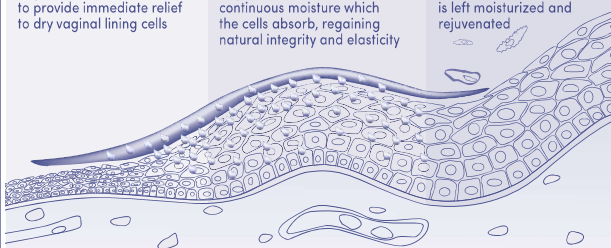

COMMONLY ASKED QUESTIONS...

How often should Replens™ Long-Lasting Vaginal Moisturizer be used?
Use once every three days as needed.

When should Replens™ Long-Lasting Vaginal Moisturizer be used?

Replens™ Long-Lasting Vaginal Moisturizer can be used any time of day or night. We recommend using the product at least 2 hours prior to intercourse to allow proper moisturization. It works best when used on a regular schedule. Because Replens™ Long-Lasting Vaginal Moisturizer provides long lasting moisture, there is no need to apply it just prior to intercourse.

Will Replens™ Long-Lasting Vaginal Moisturizer make intimacy more enjoyable?

One of the most common ways that women discover vaginal dryness is during intimacy. When used regularly, Replens™ Long-Lasting Vaginal Moisturizer helps replenish your natural vaginal moisture, making intimacy more enjoyable. The long-lasting formula provides moisture that penetrates vaginal lining cells so sexual intercourse can be more spontaneous. Since the product does not need to be applied immediately before intercourse, it does not interrupt the moment by being runny, messy or slippery. Instead, it provides long-lasting lubrication whenever the moment is right.

What causes vaginal dryness?

Nearly every woman will experience vaginal dryness sometime in her life. It is most often associated with the normal decline or fluctuation of the female hormone estrogen. This fluctuation can be triggered by childbirth, breastfeeding or menopause. Dryness can also be caused by taking certain medications, exercising intensively or being under stress. It is also common to experience vaginal dryness when douching, using tampons, or at the end of the menstrual cycle.

Can Replens™ Long-Lasting Vaginal Moisturizer be used as birth control?

No. Replens™ Long-Lasting Vaginal Moisturizer does not contain spermicides. It is not a contraceptive.

Should I use Replens™ Long-Lasting Vaginal Moisturizer during my period?

No. It is best to resume use after your flow completely stops.

Are there any side effects after using Replens™ Long-Lasting Vaginal Moisturizer?

Some women notice a residue or discharge after initial use of Replens™ Long-Lasting Vaginal Moisturizer. This is caused by the elimination of dead vaginal lining cells. Your body naturally sheds dry vaginal tissue that has built up over time. When used on a regular basis, Replens™ Long-Lasting Vaginal Moisturizer will help prevent the buildup of vaginal lining cells and the discharge should dissipate. If the discharge does not dissipate, you may wish to wait an extra day or two between applications. While use is recommended every three days, every woman is unique and you may wish to decrease the amount of time between product applications to maximize moisture and minimize discharge.

Is Replens™ Long-Lasting Vaginal Moisturizer compatible with condoms?

This product is compatible with natural rubber latex and polyisoprene condoms. This product is not compatible with polyurethane condoms.

For additional information, visit our website at: replens.com or call toll-free 1-800-833-9532 (M-F 9AM - 5PM EST)

For instructions in Spanish, please visit replens.com.

Para instrucciones en Español visite nuestro sitio de Internet en: replens.com